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**EK1T-1/3-07 «Axion»**

Single/three-channel electrocardiograph with ability to interface with pc with software of 12-channel analyzer and archiver

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Electrocardiograph EK1T-1/3-07 «AXION» is designed to acquire the electrocardiograms in manual or automatic operation modes. When in automatic operation mode, it provides simultaneous recording of 12 conventional leads: I, II, III, aVR, aVL, aVF, V1-V6. ECG acquisition duration is 3.5 sec. There is a mode of ECG recording from 3 leads simultaneously.

The signals are printed out by the thermal printing machine produced by SEIKO Company on the thermal paper with the width of 58 mm. The device plots the grid with a span of 1 mm. The leads are recorded simultaneously with measuring the heart rate, the value of which is indicated on LCD and printed out on the thermal paper.

The built-in high-quality digital filters (interference network and anti-tremulous ones) improve significantly the electrocardiogram recording.

The device is power supplied from AC mains and built-in storage battery. The device can be charged simultaneously with the cardiogram recording. The built-in storage battery supports at least 60 cardiograms without additional charging.

The device has a sound indication of the heart rate and emergency alarm. The electrocardiograph is equipped with the indicators monitoring the storage battery state and bioelectrical amplifier overloading.

Using PC allows for viewing ECG on the display in real-time mode in format of 1, 3, 6 or 12 leads simultaneously; switching on the anti-tremulous or baseline drift filters at the user's discretion; adjusting the sweep speed and scale; saving ECG in the archive or individual file with indicating the data about the patient; printing out the whole ECG or its selected part using the printer interfaced with PC. The delivery set contains the software required for joint operation of the device and PC.

Electrocardiograph EK1T-07 «AXION» meets the electrical safety requirements made for medical equipment of class II, type BF as per GOST R 50267.0-92, GOST R 50267.25-94. The device is protected against the defibrillation pulses.

The delivery set contains: the electrocardiograph, power supply unit, patient's cable, earthing cable, 4 coaptation electrodes, 6 chest suction electrodes, 2 thermal paper rolls, PC interface cable, CD with software, casing for carrying the device and accessories.

**Technical specifications:**

Input voltage range	from 0.03 up to 5 mV
Sensitivity	5; 10; 20 mm/mV
Paper speed selection	25, 50 mm/sec
Effective paper width	at least 48 mm
HR measurement range (HR)	from 30 up to 240 beats per minute
The electrocardiograph is power supplied:	
From AC mains	220 V/50 Hz
From built-in storage battery	7.2 V
Output consumed from AC mains	not more than 20 VA
Weight:	
Of the electrocardiograph	1.75 kg
Of the power supply unit	0,95 kg
Overall dimensions:	
Of the electrocardiograph	296x217x66 mm
Of the power supply unit	155x77x55 mm
Average service life	at least 5 years
3-channel printing out of simultaneous monitoring	
When operating with PC, ECG is recorded in the form of individual file for transmission via Internet	

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- **shock-proof housing.**
- **basic parameter analysis.**
- **ability to diagnose cardiac angina (ST-drift).**
- **saving the last 16 ECG examinations in archive.**

The electrocardiograph is designed to acquire the electrocardiograms (ECG) in manual or automatic operation modes from twelve conventional leads.

When in manual operation mode, the selected lead groups and service information are recorded. The average values of H-R and R-R intervals as well as the maximum and minimum R-R values and their difference are recorded at the end of this process.

When in automatic operation mode, all lead groups are recorded simultaneously by three channels. At the end of recording the results of automatic ECG parameter analysis are printed out: P wave duration, PQ, QRS, QT, QTc intervals; rotation angle of AQRS electrical axis, ST-segment drift (in case the norm is exceeded).

The electrocardiograph has an operation mode of saving ECG in its memory and can store up to 16 ECG examinations. RS-232 interface allows for connecting the electrocardiograph to the computer and transmitting ECG data recorded for analysis and storage.

There is an optional ability to transmit ECG data recorded by phone.

The device operation modes can be changed by means of the setting menu.

The electrocardiograph has automatic detection of the proper electrode application, patient's heart pacemaker (with indicating the appropriate message and pulse characters), digital filters (interference network, anti-tremulous, anti-drift).

The electrocardiograph inputs are protected against the defibrillator pulses.

The device provides recording on the thermal paper by means of the thermal printer.

The liquid-crystal character display (2 lines with 24 characters each) with highlighting is used to indicate the settings and operation mode. The device is controlled by means of a film keyboard.

The electrocardiograph is power supplied from the built-in storage battery or 220 V mains through the external power supply unit and can be automatically switched over to the mains and back. The built-in charger is automatically switched off at the end of charging. The charged storage battery can support at least 80 ECG examinations. When the device is operated, the storage battery state is monitored by the indicator located on the device housing.

The shock-proof housing has a metallic bottom and retractable aluminium handle.

Technical specifications:	
Input voltage range	from 0.03 up to 8 mV
Sensitivity	5; 10; 20 mm/mV
Paper speed selection	25; 50 mm/sec
Frequency response	from 0.05 up to 100 Hz
Input impedance	at least 10 MΩ
Common-mode rejection ratio	at least 100000
Paper width	112 mm
Resolution:	
Vertically	8 dots/mm
Horizontally	16 dots/mm
Power supply from AC mains (220±22) V	
From external power supply unit at voltage	of 15 up to 27 V
From built-in storage battery	8.4 V; 2.1 A*h
Output consumed	not more than 25 VA
Overall dimensions	260x175x70 mm
Weight	1.7 kg
Electric safety requirements	class I, type CF
Storage battery charging	not more than 4 hours
EC meets the mechanical impact requirements for group 5 as per GOST R 50444	

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**Highly Professional Electrocardiograph ECG diagnostics of human cardio-vascular system (241 diagnostic categories, analysis of 142 ECG parameters)**  
It is optimal for use by general practitioners

ЕКЗТС-04 «AXION» is a three/six/twelve-channel digital electrocardiograph (EC). When in manual operation mode, it provides simultaneous ECG data acquisition from 12 conventional leads as

well as Cabrera and Nehb's leads with real-time printing out any block consisting of 3 or 6 leads. ECG acquisition duration in automatic mode equal to 10, 16 or 24 seconds is set by the user. ECG data recorded are printed out in the format selected from the printout menu:

- 4 blocks consisting of 3 leads (I-III; aVR, aVL, aVF; V1-V3; V4-V6) with duration of 10, 16 or 24 seconds each;
- 4 blocks consisting of 3 leads (I-III; aVR, aVL, aVF; V1-V3; V4-V6) with duration of 2.5, 4 or 6 seconds each;
- 4 blocks consisting of 3 leads in Cabrera's format (aVL, I, aVR; II, aVF, III; V1-V3; V4-V6);
- 2 blocks consisting of 6 leads (I, II, III, aVR, aVL, aVF; V1-V6) with duration of 5, 8 or 12 seconds each;
- 12 leads in a compact way across the paper.

It is possible to reprint out ECG data recorded earlier in other format with adjusted sensitivity and speed parameters. The electrocardiograph provides five operation modes – manual, automatic, rhythm, archive and setting. The operation

**EK3TC-04 «Axion»**

Three-channel electrocardiograph

mode parameters are set in five main menu sections to be switched over by means of the keys on the device panel. When powered on, the device and extractor are automatically self-tested. Prior to each examination the device automatically detects the proper application, possible electrode confusion and induced noise level with indicating the appropriate message on LCD.

The device provides full complex electrocardiographic diagnostics of human cardio-vascular system without using PC for the first time in Russia.

Electrocardiograph EK3TC-04 «Axion»:

1. It automatically measures 142 amplitude-time ECG parameters.
2. It forms typical cardiac cycles with the waves marked.
3. When using the thermal paper without any grid, it plots the grid with a span of 1 or 5 mm to print out ECG and typical cardiac cycles.
4. It automatically forms the syndrome-based diagnostic decision on ECG basis for adults and children at the age of 1 year and older with high efficiency (sensitivity and specificity of diagnosing the basic electrocardiographic syndromes comprise at least 95%). Number of diagnostic categories is 241.
5. It makes the statistic analysis of the heart rate with printing out the parameters measured and provides plotting the cardiac intervalograms, R-R intervals scatterograms and histograms (C version).
6. It allows for monitoring the heart rate with arrhythmia diagnosing.
7. It detects available heart pacemaker of a patient.
8. It prints out all parameters listed above.

The device allows for saving ECG and rhythmograms recorded in the built-in archive with the memory for 99 examinations, which is very convenient for use at sick calls. The device is controlled by a simplified for use drip-proof keyboard by means of the opening menu system. The data on the operation modes and preliminary settings are indicated on a large 2-line alphanumeric LCD. The built-in real-time clocks and calendar allow for registering the exact time of examination.

The device has efficient digital network interference, anti-tremulous and baseline drift filters to be switched on at the user's discretion in any combination.

In order to print ECG out on the thermal paper the thermal printing machine is used providing convenient thermal paper injection and ECG recording with resolution of 8 dots/mm.

The electrocardiograph inputs are protected against defibrillator pulses with output of up to 360 J.

The electrocardiograph is power supplied from both the mains and rechargeable storage battery.

The fully charged storage battery supports at least 60 complete ECG examinations. The built-in automatic charger allows for monitoring the battery state of charge and recharging it as necessary excepting for its overcharging. The electrocardiograph is manufactured in versions D and C differing from each other in their functionality. The children's version is also available. The device has design patent 59613.

Technical specifications:	
Input voltage range	from 0.03 up to 10 mV
Sensitivity	2.5; 5; 10; 20; 40 mm/mV
Paper width	112 mm
Paper speed selection	5; 12.5; 25; 50 mm/sec
Time constant	at least 3.2 sec.
Input impedance at signal frequency of 10	at least 20 MOhm
Common-mode rejection ratio	at least 100 000
Attenuation of interference from the mains, when the rejection filter is switched on	at least 12 dB
Output consumed from AC mains	not more than 50 VA
Battery charging time	not more than 5 hours
Overall dimensions of EC without extractor:	
- Length	260 mm
- Width	195 mm
- Height	65 mm
Weight	not more than 2.0 kg
USB interface for connecting with PC	
EC meets the electric safety requirements made for devices of class I, type CF as per GOST R 50267.0-92, GOST R 50267.25-94.	
EC meets the mechanical impact requirements for group 5 as per GOST R 50444	
Device provides recording of leads I, II, III, aVR, aVL, aVF, V1-V6, D, A, I	
EC is power supplied:	
— From rechargeable storage battery at rated voltage of 12 V, 2 A·h	
— From AC mains at voltage of (220±22) V and rated frequency of 50 Hz through external power supply unit	

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When in automatic operation mode, all lead groups are recorded simultaneously by three or six channels. At the end of recording the results of automatic ECG parameter analysis are printed out: P wave duration, PQ, QRS, QT, QTc intervals; rotation angle of AQRS electrical axis, ST-segment drift.

The electrocardiograph has an operation mode of saving ECG in its memory and can store up to 100 ECG examinations. USB interface allows for connecting the electrocardiograph to the computer and transmitting ECG data recorded for analysis and storage. There is an ability to transmit ECG data recorded by phone. The device operation modes can be changed by means of the setting menu.

The electrocardiograph has automatic detection of the proper electrode application, patient's heart pacemaker (with indicating the appropriate message and pulse characters), digital filters (interference network, anti-tremulous, anti-drift). The electrocardiograph inputs are protected against the defibrillator pulses.

The data are recorded on the thermal paper with the width of 112 mm from three or six leads. It is possible to set the automatic grid plotting.

The graphic display with highlighting is used to indicate the settings and operation mode. Three, six or twelve ECG leads are indicated on the display simultaneously. The device is controlled by means of a film keyboard.

The electrocardiograph is power supplied from the rechargeable storage battery (NiCd; 9.6 V; 2.0 A\*h) or 220

V mains through the external power supply unit. The built-in charger is automatically switched off at the end of charging. The charged storage battery can support at least 80 ECG examinations. When the device is operated, the storage battery state is monitored by the indicator located on the front panel of the device.

**Technical specifications:**

Input voltage range	from 0.03 up to 10 mV
Sensitivity	12.5; 25; 50 mm/sec
Paper width	112 mm
Paper speed selection	5; 12.5; 25; 50 mm/sec
Sampling frequency	8000 Hz/channel
Input impedance	at least 10 MΩ
Common-mode rejection ratio	at least 100 000
Frequency response	from 0.05 up to 100 Hz
Display: CRT size resolution	10 cm 320x240 dots
Resolution: Vertically Horizontally	8 dots/mm 16 dots/mm
Output consumed	not more than 25 VA
Power supply: - From AC mains - From external DC power supply unit at voltage - From rechargeable storage battery	(220±22) V of 15 up to 27 V 9.6 V; 2.0 A*h
Overall dimensions	260x195x65 mm
Weight	1.7 kg
Electrical safety requirement	class I, type CF

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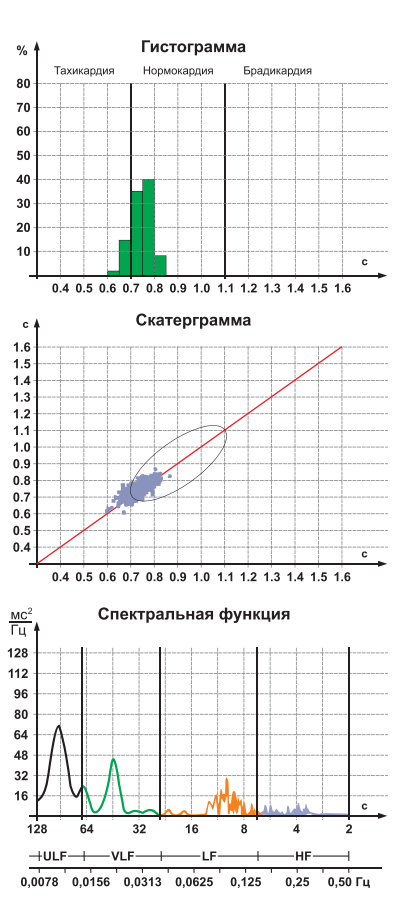
## Complex «Varicard-Axion»

Cardiographic complex for heart rate variability analysis

Complex «Varicard-Axion» is created specially for equipping the general (family) practitioner's workplace.

It contains:

- Three-channel electrocardiograph EK3TC-04 «AXION»;
- Unit for cardiac intervalogram processing and heart rate variability analysis «Varicard 2.51»;
- Portable computer, Samsung laptop;
- Samsung printer A4;
- Software at CD.

This complex resolves the objectives on assessing the adapting capabilities of the body based on the heart rate variability analysis and electrocardiogram acquisition, its complete automatic interpretation with measuring the amplitude-time parameters and issuing the preliminary diagnostic decision.

EK3TC-04 «AXION» is a three/six/twelve-channel electrocardiograph. When in manual operation mode, it provides ECG data acquisition from 12 conventional leads as well as Cabrera and Nehb's leads with real-time printing out any block consisting of 3, 6 or 12 leads across the paper. ECG acquisition duration in automatic mode equal to 10, 16 or 24 seconds is set by the user. ECG data recorded are printed out in the format selected from the printout menu. The device provides:

- Automatic measuring 142 amplitude-time ECG parameters;
- Forming typical cardiac cycles with the waves marked;
- Plotting the grid with a span of 1 or 5 mm, when using the thermal paper without any grid;
- Automatic forming the syndrome-based diagnostic decision with high efficiency. Number of diagnostic categories is 241;
- Monitoring the heart rate with arrhythmia diagnosing;
- Printing out all parameters on the paper with the width of 112 mm.

The data on the operation modes and preliminary settings are indicated on alphanumeric LCD. The built-in real-time clocks and calendar allow for registering the exact time of examination. The electrocardiograph is power supplied from both 220 V mains and rechargeable storage battery. The device is connected with PC through USB interface. The complex contains the software at CD.

«Varicard 2.51» is a unit entering the electric cardiac signals from the electrodes into the personal computer for processing by means of the software, which is a part of the complex. In the course of processing the objective is resolved on assessing the adaptive capabilities of the body based on the heart rate variability (HRV) analysis. In order to analyse HRV the methods recommended by both the European-American standards and the Russian methodical recommendations are implemented. More than 30 different HRV parameters are assessed.

Basic analysis methods:

- Histogram plotting (heart rate variation);
- Autocorrelation analysis;
- Correlation rhythmography;
- Spectral analysis.

All graphs of the functions specified can be reviewed on PC display. In addition to that, when analysing the data, the rate of strain in the regulatory systems is assessed, which allows for making the objective judgement on the adaptive capabilities and functional reserves of the body. The scheme of "Traffic Lights" type is used to provide visual demonstration of the functional state assessment results based on the heart rate variability analysis data. It provides GREEN, YELLOW and RED zones of states, which characterize the normal state of a human being accordingly, his prenosological state (a transition state between health and illness) and prepathological state requiring attention of a doctor. The final document of the complex is a patient's card - health passport to be printed out by means of the colour printer.

The software, which is a part of the complex set, also implements all abilities of electrocardiograph EK3TC-04 «AXION» interfaced with PC and allows for forming the patients' data base.

Complex «Varicard-Axion» can be indispensable in the following areas of its application:

- Screening-examinations during scheduled health survey;
- Screening-diagnosing of opiate addiction;
- Preventive monitoring during scheduled health survey;
- Medical diagnosing in the clinical outpatient department;
- Profound clinical and scientific researches;
- Monitoring during rehabilitation and secondary prevention;
- Studying the effects of medication and physiotherapy;
- Selection of adequate physical load in sports and recreational physical exercises;
- Assessment of the body's state and functional resources in emergency situations.

The technology is intended for use by the general practitioners, physicians, infectious disease specialists, psychiatrists, narcologists, rehabilitation medicine physicians, oncologists, professional recruitment specialists.